## Peripherally Inserted Central Catheter (PICC) Patient Information Booklet

**Patient's Name:** \_\_\_\_\_  
\_\_\_\_\_

**Doctor's Name and Telephone Number:** \_\_\_\_\_  
\_\_\_\_\_

**IV Nurse's Name and Telephone Number:** \_\_\_\_\_  
\_\_\_\_\_

**Home Care Agency:** \_\_\_\_\_

**Telephone Number:** \_\_\_\_\_

**Contact Name:** \_\_\_\_\_

**Catheter Information:** \_\_\_\_\_

**Date Inserted:** \_\_\_\_\_

**Inserted by:** \_\_\_\_\_

**Lot Number:** \_\_\_\_\_

**Catheter Length:** \_\_\_\_\_

### Introduction:

Your doctor has requested that a PIC catheter be inserted for your medical treatment. This booklet has been given to you to provide information about the Arrow PIC catheter. Use it in addition to the directions you have received from your nurse or doctor.

PIC stands for a “peripherally inserted central” catheter. It is a small tube that is placed into a blood vessel in one of your arms and the tip is advanced to a position that is close to your heart (refer to Fig. 1). Medications or nutritional fluids can be given through the catheter for an illness that requires treatment over a period of time. A PIC catheter is an alternative to having repeated needle sticks or surgery to place another kind of IV (intravenous) device.

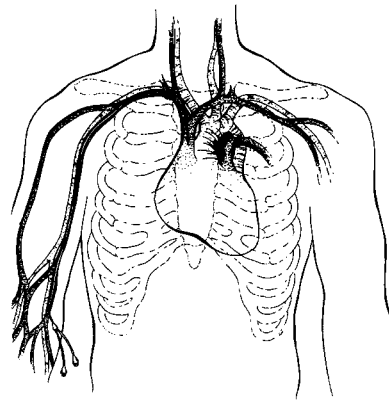
Fig. 1

The PIC catheter consists of a long, hollow tube (lumen) with one (or more) holes at the tip that lies within the body and a hub at the other end that is outside of the body on the arm surface (refer to Fig. 2). Medications or fluids can be given through the hub continuously or injected periodically.

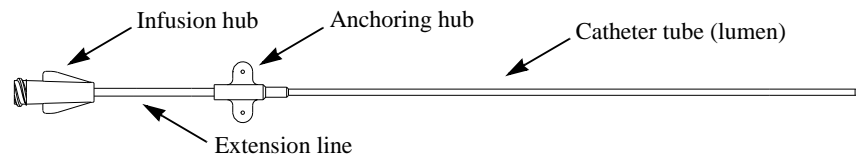
Fig. 2

### Catheter Insertion:

The catheter is inserted by a doctor or nurse into a blood vessel on the inside of your arm near where the elbow bends. Measurements are taken to estimate the amount of catheter that needs to be inserted. An x-ray may be taken to provide a picture of where the catheter tip lies within your body. The doctor or nurse will use sterile technique to insert the catheter. This means that precautions will be taken to make sure that germs are not inserted with the catheter. Similar care should be taken with all further catheter-related procedures.

### Catheter Care and Maintenance:

#### Dressings:

The insertion site and external portion of the PIC catheter should be covered with a protective dressing. The dressing will help to hold the catheter in place and keep the area clean and protected from germs.

Because catheter is small and pliable, the catheter body can be anchored in place by using stitches, Steri-Strip®\* or a special anchoring device. The special anchoring device consists of a catheter clamp and a crescent-shaped adhesive strip. The strip is

placed on the arm and the catheter clamp that has been applied to the catheter is snapped into place (refer to Figs. 3, 4, 5). Another dressing, either gauze or a transparent sheet, is placed over the site as a final protective measure.

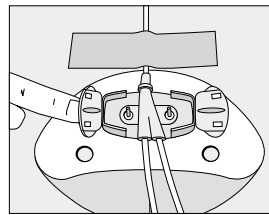
Fig. 3

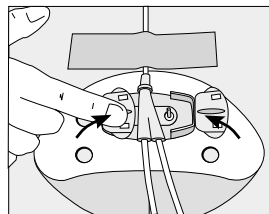
Fig. 4

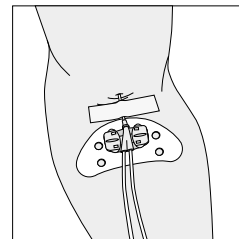
Fig. 5

If you or a family member have been trained to redress your catheter, follow the directions given by your doctor or nurse. Be

sure to wash your hands prior to starting the procedure and use sterile technique as you have been instructed. As a safeguard, do not use sharp instruments such as scissors during the dressing change to avoid accidentally cutting your catheter. Inspect the insertion site regularly for any redness or drainage, and watch the catheter to make sure it does not move in or out further than when it was inserted. If the dressing becomes loose, make sure that it is changed.

**Flushing:**

To keep your PIC catheter open to flow, it may be flushed and filled with a weak sterile salt solution also known as sterile

normal saline or an anti-clotting solution. The solution keeps blood from backing up into the lumen and forming a clot. If you or a family member are caring for your catheter, follow the directions given by your doctor or nurse. They will tell you what solution should be used as well as the amount that should be given and how often the catheter should be flushed.

**Complications:**

Although you should not expect a problem to occur that is related to your PIC catheter, it is important that you become familiar with the signs and symptoms that might signal such a problem.

\*Steri-Strip® is a registered trademark of 3M HealthCare.

Signs and Symptoms	Possible Cause	Directions	Prevention
Drainage, redness, pain, swelling around insertion site, fever and chills.	Infection.	Call doctor or IV nurse.	Use sterile technique. Keep sterile dressing over site. Wash hands prior to procedures.
Arm or shoulder swelling, swishing in ear on same side of body where catheter is located while medication given.	Catheter position change.	Call doctor or IV nurse. Do not inject any solutions into catheter until talking with doctor or nurse.	Inject flushing/locking solution slowly.

Signs and Symptoms	Possible Cause	Directions	Prevention
Inability to inject.	Catheter clotted or kinked.	Call doctor or IV nurse.	Completely fill catheter lumen with locking solution between treatments. Flush catheter well before and after medications.
Leaking from external catheter.	Break in catheter material, hub separation.	Call doctor or IV nurse. Fold catheter together below leaking area and tape securely.	Do not use alcohol or acetone (as in nail polish or tape remover) on catheter. Do not pull on catheter.
Pain on injection.	Inflammation of vein.	Call doctor or IV nurse.	Medications should be given slowly.

**Questions:**

The following questions cover topics concerning your daily routine while your PIC catheter is in place. You may want to discuss them with your doctor or nurse.

Are there any limits to my activities?

---

---

---

---

May I bathe/shower and swim while my catheter is in place?

---

---

---

---

How often should my dressing be changed?

---

---

---

---

How often should my catheter be flushed and what solution should be used?

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

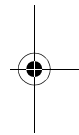
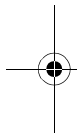
Other:

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

What fluids and medications will I be receiving through my catheter?

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

Are there any special instructions concerning my IV medications?

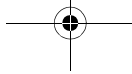
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

Who should I call if I have questions about my PIC catheter?

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