

UROLIFT™

Patient Implant Card

Patient Name

Physician Name

Procedure Date



NeoTract™, Inc.
4155 Hopyard Road
Pleasanton, CA 94588 USA
877.408.9628, +1 925.401.0700
uroliftpatientsupport@teleflex.com
www.urolift.com

Affix
Label(s)
Here

Non-clinical testing has demonstrated that the UroLift® Implant is MR Conditional. A patient with this device can be safely scanned in an MR system immediately after placement meeting the following conditions:

- Static magnetic field of 3.0 Tesla or less
- Maximum spatial gradient magnetic field of 1,500 Gauss/cm (15 T/m) (extrapolated)
- Maximum MR system reported, whole body averaged specific absorption rate (SAR) of 4 W/kg for 15 minutes of scanning (i.e., per pulse sequence) (First Level Controlled Operating Mode)

Under the scan conditions defined above, the UroLift Implant is expected to produce a maximum temperature rise of 2.4°C after 15 minutes of continuous scanning (i.e., per pulse sequence).

In non-clinical testing, the image artifact caused by the UroLift Implant extends approximately 15 mm from the UroLift Implant when imaged with a gradient echo pulse sequence and a 3.0 Tesla MRI system.