Long-Term Hemodialysis Catheter - Antegrade (VectorFlow®) Patient Information Booklet

Introduction:

Your doctor has requested that a long-term hemodialysis catheter be inserted for your medical treatment. This booklet has been given to you to provide information about the dialysis catheter. Use it in addition to the directions you have received from your nurse or doctor.

The dialysis catheter consists of a long, hollow tube (lumen) that provides access to a central blood vessel. The parts outside the body are designed to allow access to dialysis machines and can also be used by clinicians to administer certain medications (refer to Figure 1).

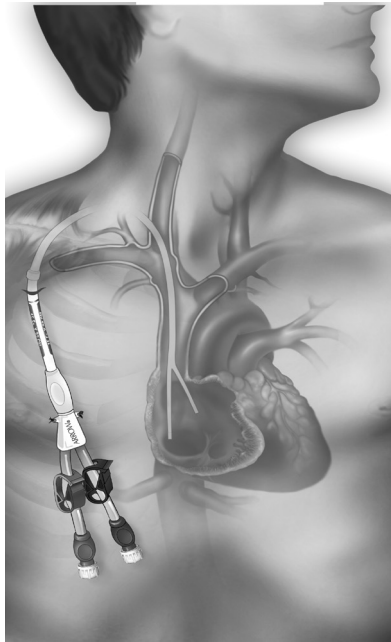
Figure 1

Catheter Care and Maintenance:

Dressings:

The insertion site and external portion of the dialysis catheter should be covered with a protective dressing. The dressing will help to hold the catheter in place and keep the area clean and protected from germs.

The catheter will be initially anchored in place by using stitches or sterile tape strips. A transparent dressing is placed over the site as a final protective measure.

If you or a family member have been trained to redress your catheter, follow the directions given by your doctor or nurse. Be sure to wash your hands prior to starting the procedure and use sterile technique as you have been instructed. As a safeguard, do not use sharp instruments such as scissors during the dressing change to avoid accidentally damaging your catheter. Inspect the insertion site regularly for any redness or drainage, and watch the catheter to make sure it does not move in or out further than when it was inserted. If the dressing becomes loose, make sure that it is changed. Avoid tension or tugging on the catheter. If your catheter is accidentally removed, immediately cover the site with a transparent dressing and contact your nurse or doctor.

Flushing:

If you or a family member are caring for your catheter, follow the directions given by your doctor or nurse. They will tell you what solution should be used as well as the amount that should be given and how often the catheter should be flushed.

Complications:

Although you should not expect a problem to occur that is related to your dialysis catheter, it is important that you become familiar with the signs and symptoms that might signal such a problem. See the following table.

| Signs and Symptoms | Possible Cause | Directions | Prevention |
|---|---------------------------------------|---|--|
| Drainage, redness, pain, swelling around insertion site, fever and chills | Infection | Call doctor or nurse | Use sterile technique Keep sterile dressing over site Wash hands prior to procedures |
| Catheter leaking | Catheter damaged or catheter uncapped | Call doctor or nurse Fold catheter together below leaking area and tape securely | Do not use alcohol or acetone (as in nail polish or tape remover) on catheter Do not pull on catheter |
| Pain at site | Inflammation of vein | Call doctor or nurse | Medications should be given slowly |

Frequently asked Questions:

The following questions cover topics concerning your daily routine while your dialysis catheter is in place. You may want to discuss these and other topics with your doctor or nurse.

- ***May I bathe and swim while my catheter is in place? It is recommended that you do not submerge your catheter site in water. Showering is permitted provided a waterproof dressing is in place. If the catheter site becomes soaked, pat the site dry with a clean towel and have your dressing changed immediately.***

- ***How often should my dressing be changed? Initially your dressing should be changed weekly, it should be changed immediately if it becomes soiled or wet.***

- ***How often should my catheter be flushed and what solution should be used? Your catheter should be flushed before and after each use at the healthcare facility performing your dialysis procedure.***

Notes:

Symbol Glossary: Symbols are in compliance with ISO 15223-1.
Some symbols may not apply to this product. Refer to product labeling for symbols that apply specifically to this product.

| | | | | | | | |
|--------------|------------------------|--------------|----------------|--------------|-----------------------------|------------|--------------------------|
| | | | | | | | |
| Patient Name | Healthcare Institution | Implant Date | Medical Device | Manufacturer | Patient Information Website | Lot Number | Unique Device Identifier |

Arrow, the Arrow logo, VectorFlow, Teleflex and the Teleflex logo are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. © 2021 Teleflex Incorporated. All rights reserved.

Qualitative and quantitative information on materials and substances to which patients can be exposed:

| % | Material/Substance | Role/Notes |
|---------|------------------------|--|
| balance | Polyurethanes | Catheter body and extension line |
| 6 - 11% | Barium Sulfate | Radiopacity Agent |
| <0.2% | Polyester | Cuff |
| <0.1% | Polymethylmethacrylate | Printing |
| <0.1% | Colorants | FDA 21CFR Exempt (Safe for Medical Device Use) |

Trajni kateter za hemodijalizu – antegradni (VectorFlow®) Knjižica s informacijama za pacijente

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**Arrow International LLC**