
# Long-Term Hemodialysis Catheter - Retrograde (VectorFlow®) Patient Information Booklet

---

## Introduction:

Your doctor has requested that a long-term hemodialysis catheter be inserted for your medical treatment. This booklet has been given to you to provide information about the dialysis catheter. Use it in addition to the directions you have received from your nurse or doctor.

The dialysis catheter consists of a long, hollow tube (lumen) that provides access to a central blood vessel. The parts outside the body are designed to allow access to dialysis machines and can also be used by clinicians to administer certain medications (refer to Figure 1).

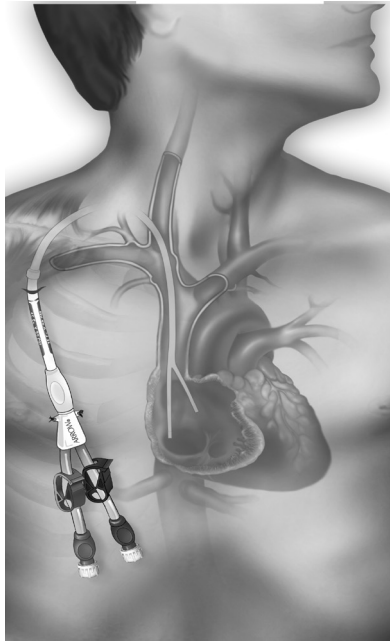*Figure 1*

---

## Catheter Care and Maintenance:

### Dressings:

The insertion site and external portion of the dialysis catheter should be covered with a protective dressing. The dressing will help to hold the catheter in place and keep the area clean and protected from germs.

The catheter will be initially anchored in place by using stitches or sterile tape strips. A transparent dressing is placed over the site as a final protective measure.

If you or a family member have been trained to redress your catheter, follow the directions given by your doctor or nurse. Be sure to wash your hands prior to starting the procedure and use sterile technique as you have been instructed. As a safeguard, do not use sharp instruments such as scissors during the dressing change to avoid accidentally damaging your catheter. Inspect the insertion site regularly for any redness or drainage, and watch the catheter to make sure it does not move in or out further than when it was inserted. If the dressing becomes loose, make sure that it is changed. Avoid tension or tugging on the catheter. If your catheter is accidentally removed, immediately cover the site with a transparent dressing and contact your nurse or doctor.

## Flushing:

If you or a family member are caring for your catheter, follow the directions given by your doctor or nurse. They will tell you what solution should be used as well as the amount that should be given and how often the catheter should be flushed.

## Complications:

Although you should not expect a problem to occur that is related to your dialysis catheter, it is important that you become familiar with the signs and symptoms that might signal such a problem. See the following table.

Signs and Symptoms	Possible Cause	Directions	Prevention
Drainage, redness, pain, swelling around insertion site, fever and chills	Infection	Call doctor or nurse	Use sterile technique Keep sterile dressing over site Wash hands prior to procedures
Catheter leaking	Catheter damaged or catheter uncapped	Call doctor or nurse Fold catheter together below leaking area and tape securely	Do not use alcohol or acetone (as in nail polish or tape remover) on catheter Do not pull on catheter
Pain at site	Inflammation of vein	Call doctor or nurse	Medications should be given slowly

## Frequently asked Questions:

The following questions cover topics concerning your daily routine while your dialysis catheter is in place. You may want to discuss these and other topics with your doctor or nurse.

- ***May I bathe and swim while my catheter is in place? It is recommended that you do not submerge your catheter site in water. Showering is permitted provided a waterproof dressing is in place. If the catheter site becomes soaked, pat the site dry with a clean towel and have your dressing changed immediately.***  
\_\_\_\_\_
- ***How often should my dressing be changed? Initially your dressing should be changed weekly, it should be changed immediately if it becomes soiled or wet.***  
\_\_\_\_\_
- ***How often should my catheter be flushed and what solution should be used? Your catheter should be flushed before and after each use at the healthcare facility performing your dialysis procedure.***  
\_\_\_\_\_

## Notes:

---

---

---

Symbol Glossary: Symbols are in compliance with ISO 15223-1. Some symbols may not apply to this product. Refer to product labeling for symbols that apply specifically to this product.

Patient Name	Healthcare Institution	Implant Date	Medical Device	Manufacturer	Patient Information Website	Lot Number	Unique Device Identifier

Arrow, the Arrow logo, VectorFlow, Teleflex and the Teleflex logo are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. © 2021 Teleflex Incorporated. All rights reserved.

### MR Conditional Advisory:

Non-clinical testing demonstrated that the Arrow-Clark VectorFlow Retrograde Chronic Hemodialysis Catheter (CHDC) is MR Conditional. A patient with this device can be safely scanned in an MR system meeting the following conditions:

- Static magnetic field of 1.5 Tesla and 3.0 Tesla only
- Maximum spatial gradient magnetic field of 4,000 Gauss/cm (40 T/m) or less
- Maximum MR system reported, whole body averaged specific absorption rate (SAR) of 4 W/kg (First Level Controlled Operating Mode)

Under the scan conditions defined above, the Arrow-Clark VectorFlow Retrograde CHDC is not expected to produce any MRI-related heating.

In non-clinical testing, the image artifact caused by the Arrow-Clark VectorFlow Retrograde CHDC extends approximately 5 mm from the device when imaged with a gradient echo pulse sequence in a 3.0T MRI system.

#### Qualitative and quantitative information on materials and substances to which patients can be exposed:

%	Material/Substance	Role/Notes
balance	Polyurethanes	Catheter body, Juncture Hub, and extension lines
6.3 - 10.3%	Barium Sulfate	Radiopacity Agent
<0.2%	Polyester	Cuff
<1%	Green Silicone	Compression Sleeve
2.4 - 3.4%	304 Stainless Steel	Juncture Hub Cannula
<0.1%	Polymethylmethacrylate	Printing
<0.1%	Colorants	FDA 21CFR Exempt (Safe for Medical Device Use)

# Trajni kateter za hemodijalizu – retrogradni (VectorFlow®) Knjižica s informacijama za pacijente

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**Arrow International LLC**