


PROCEDURE GUIDE – GENERAL SURGERY


Traditional Lap Chole

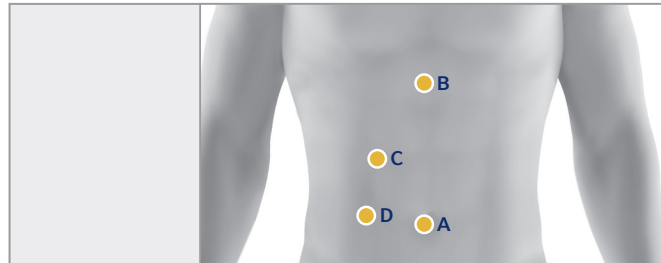

PORT / INSTRUMENT

PORTS: 5 mm or 10 mm ports for scope / instrumentation

MiniLap Chole

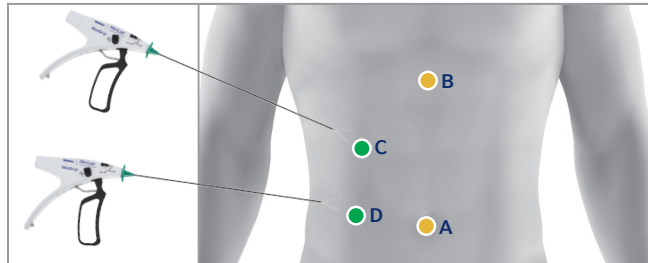


PORT / INSTRUMENT

A / B PORTS: 5mm or 10mm ports for scope / instrumentation

C / D PORTS: MiniGrip® Alligator Grasper for manipulation and retraction

2x3 MiniLap Chole & MiniPolar

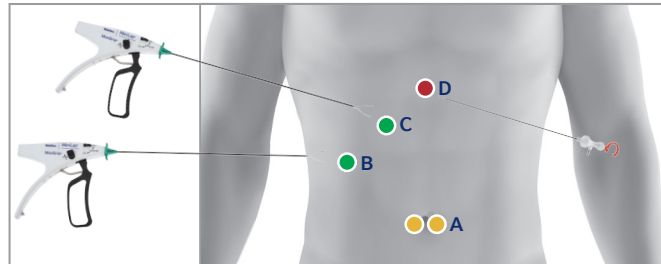

PORT / INSTRUMENT


INSTRUMENTS: Scope, Clip Applier, Bag

A PORT: 5mm or 10mm port for instrumentation

B / C PORTS: MiniGrip® Alligator Grasper for manipulation and retraction

D PORTS: MiniPolar™ Straight Spatula Probe for cautery and dissection

2 x 2 MiniLap Appendectomy

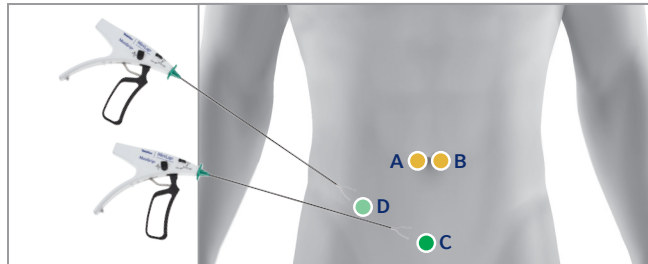


PORT / INSTRUMENT

A / B PORTS: 5mm or 10mm ports for scope / instrumentation

C PORT: MiniGrip® Alligator Grasper for manipulation of appendix


D PORT: MiniGrip® Alligator Grasper, alternate to C Port

PORT COMPARISON

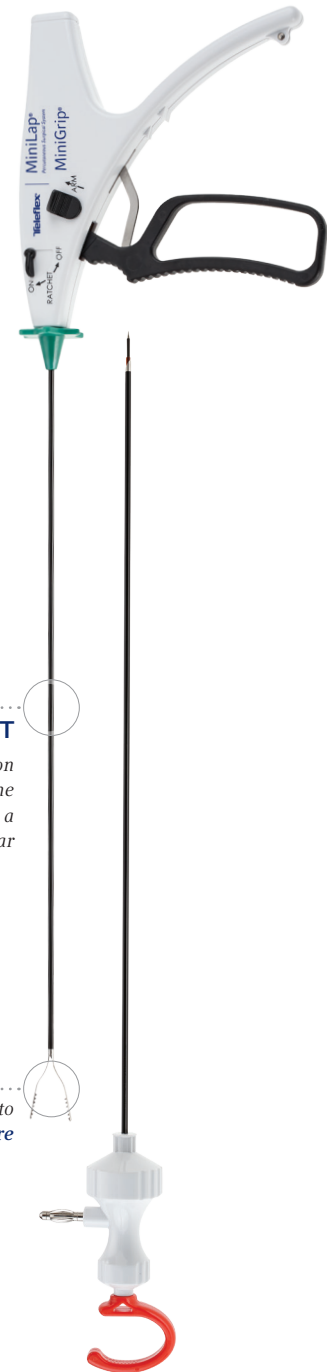

PROCEDURE GUIDE – GYNECOLOGICAL SURGERY


2 Port Advanced Gynecological Procedures Oophorectomy, Hysterectomy, Myomectomy

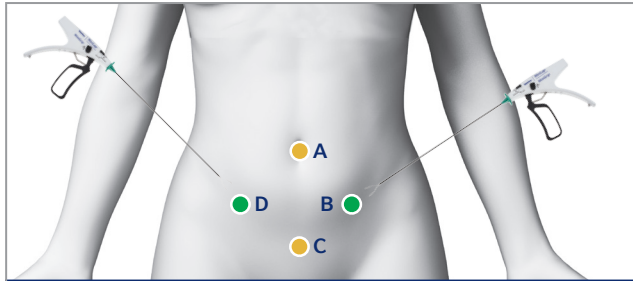

PORT / INSTRUMENT

- A PORT:** 5mm or 10mm port for scope
- B / D PORTS:** MiniGrip® Alligator Grasper for manipulation and retraction
- C PORT:** Energy Device and instrumentation


1x2 MiniLap Excision of Endometriosis/Adhesiolysis

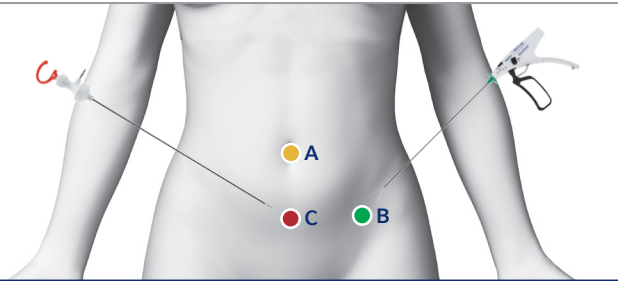

PORT / INSTRUMENT

- A PORT:** 5mm or 10mm port for scope
- B PORTS:** MiniGrip® Alligator Grasper for manipulation and retraction
- C PORT:** Electrosurgery MiniPolar Probe used for ablation of endometriosis


Laparoscopic Ovarian Cyst Removal

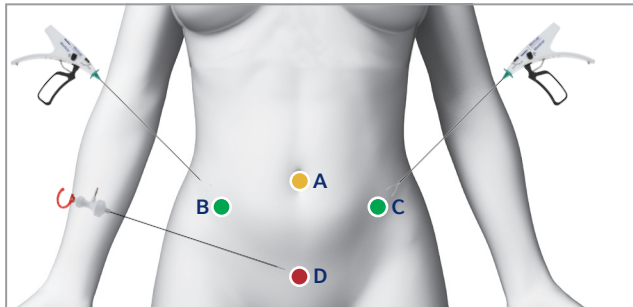

PORT / INSTRUMENT

- A PORT:** 5mm or 10mm port for scope
Optional second port for instrumentation
- B / C PORT:** MiniGrip® Alligator Grasper for manipulation and retraction
- D PORT:** MiniPolar™ Straight Spatula Probe used for cautery and dissection


Tubal Ligation

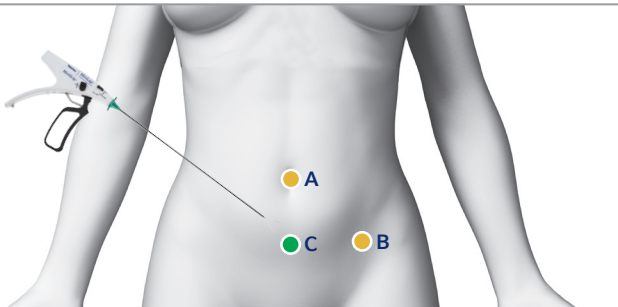


PORT / INSTRUMENT

- A / B PORTS:** 5mm or 10mm ports for scope/instrumentation
- C PORT:** MiniGrip® Alligator Grasper used to grasp fallopian tubes


MINILAP® SYSTEM WITH MINIGRIP® HANDLE

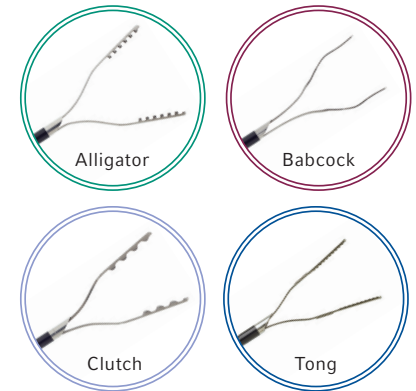

MINILAP® PERCUTANEOUS GRASPERS

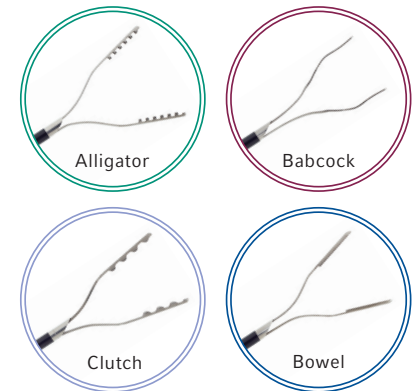

MINIPOLAR™ ELECTROSURGERY PROBES

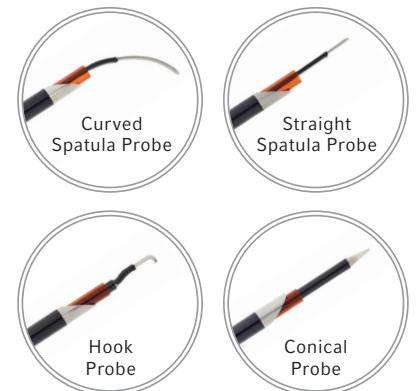

TO ORDER, CALL TELEFLEX CUSTOMER SERVICE AT +1.919.433.4999

Teleflex, MiniGrip, MiniLap and MiniPolar are trademarks or registered trademarks of Teleflex Incorporated or its affiliates.
© 2015 Teleflex Incorporated. All rights reserved. LA MC-000827 Rev 1

Teleflex®

TELEFLEX LATIN AMERICA 3015 Carrington Mill Boulevard Morrisville, NC 27560 USA
For product information please contact la.cs@teleflex.com or +1.919.433.4999
TELEFLEX.COM