

PIGGYBACK[®]

wire converter

Convert 0.014"
Procedures

Into 0.035"
Possibilities




vascular
SOLUTIONS

PIGGYBACK[®]

wire converter

Transition without Losing Position

The PiggyBack is a 0.035" outer diameter wire converter that can be delivered over and locked onto a 0.014" guidewire to allow subsequent delivery of 0.035" compatible catheters, sheaths and devices.

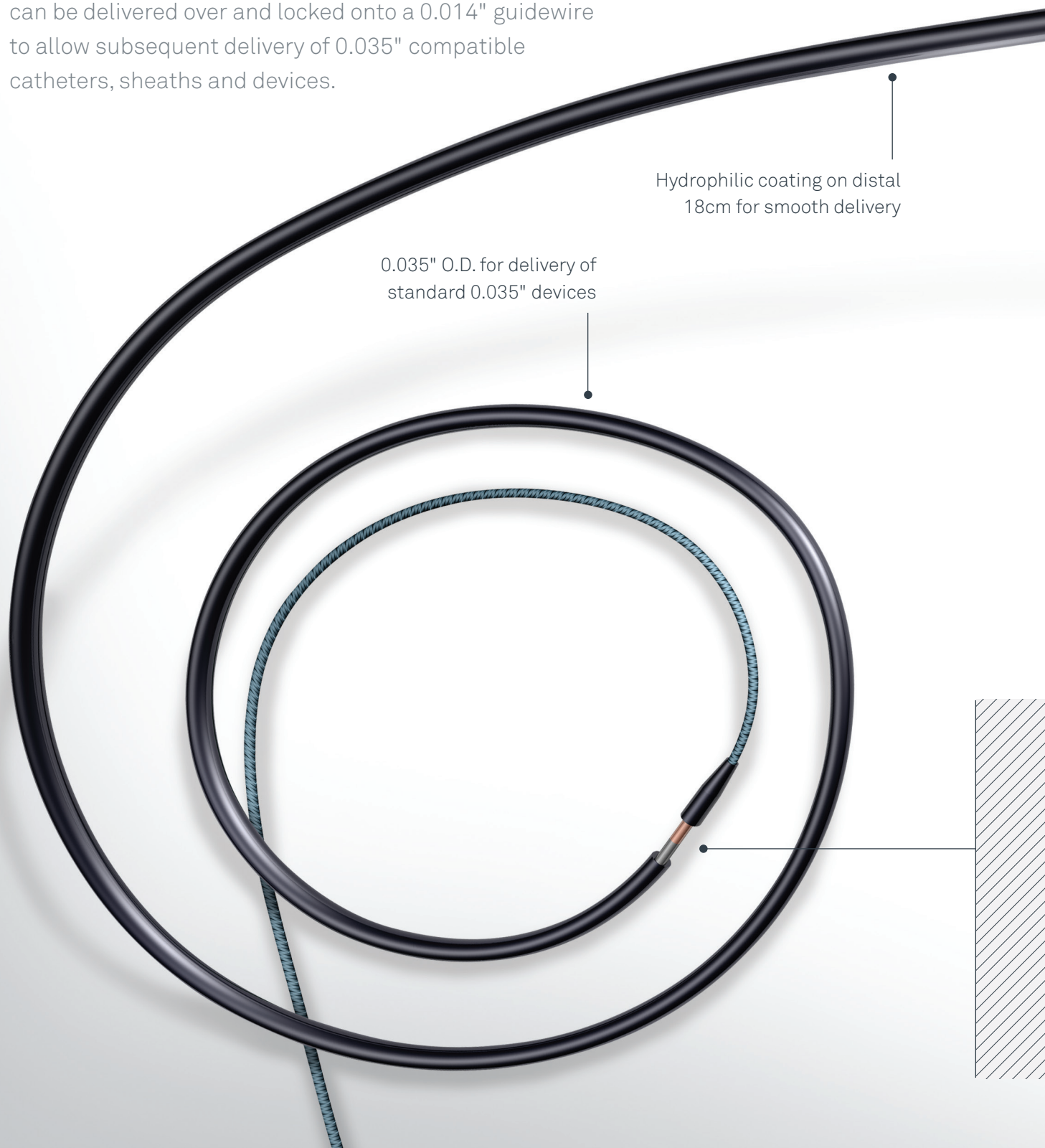




0.014" Inside, 0.035" Outside



The PiggyBack is useful for peripheral procedures requiring a 0.014" guidewire for easier access and a 0.035" guidewire for more support. The PiggyBack provides the ability to convert a 0.014" guidewire to a 0.035" platform without losing vessel placement or exchanging guidewires.

- For treating different sized vessels during the same procedure
- For crossing tortuous anatomy

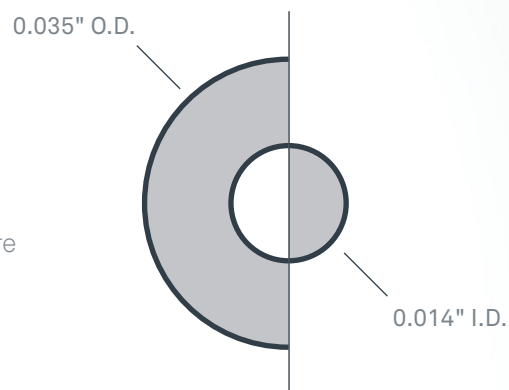

How Does It Work?

The PiggyBack can be delivered over a 0.014" guidewire *in situ* or preloaded onto a 0.014" guidewire before it is placed intravascular.

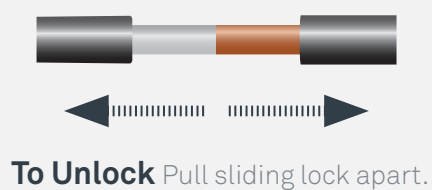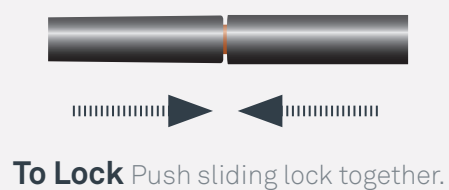


PiggyBACK[®]

wire converter

The PiggyBack is intended to be used with guidewires to access the peripheral vasculature and to facilitate placement of interventional devices.

Model	Length	Required Guidewire Diameter	Required Guidewire Minimum Length For Exchange	Minimum Introducer Size	Distal Hydrophilic Coating Length
8802	145cm	0.014"	290cm	4F (0.053" I.D.)	18cm

Packaged in quantities of 1 unit per box.

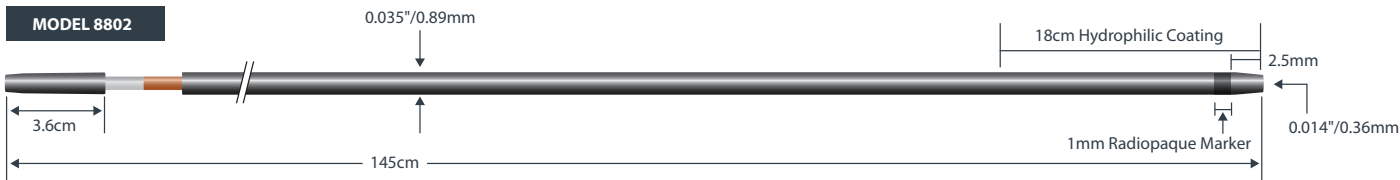

Please see the Instructions for Use for a complete listing of the indications, contraindications, warnings and precautions.

CAUTION: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

PiggyBack is a registered trademark of Vascular Solutions, Inc.

©2016 Vascular Solutions, Inc. All rights reserved. ML2120 Rev. D 06/16

This literature is intended for distribution only in the U.S.



Vascular Solutions, Inc.
6464 Sycamore Court North
Minneapolis, Minnesota 55369 USA

Customer Service: 888.240.6001
customerservice@vasc.com
www.vasc.com