



## **RUSCH® FILIFORMS AND FOLLOWERS**

Your solution for a number of urethral procedures

### **URETHRAL DILATION**

Urethral dilation may be needed for a number of reasons including false passages, urethral trauma, strictures, enlarged prostate, or previous surgeries. Filiforms and Followers can be used to negotiate the compromised urethra and eliminate the need for more invasive procedures such as cystoscopy and suprapubic access when access to the bladder is necessary.

### **FILIFORMS**

Filiforms are smaller than urethral catheters and more easily passed through a compromised urethra. Followers are then used to dilate the urethra to a size suitable for passage of a urethral catheter when needed. Filiforms and Followers are also used in the treatment of urethral stricture disease and post-operative therapy. In these cases, placement of a catheter is not always required. Chronic strictures can be managed with repeat dilations.

### **FILIFORMS AND FOLLOWERS**

Teleflex® Rusch® Filiforms and Followers are disposable, helping to prevent the risk of cross-contamination and are medical grade soft plastic with universal fittings. Filiforms are available in both straight and spiral tips and are 33 cm (appx.13") long. Followers are available in Bougie (no drainage eyes) or in Catheter (allowing bladder drainage) styles. The smaller sizes 8 and 10 Fr. are 34 cm (13.5"); the larger sizes are 36 cm (14.5"). All are available in boxes of 10.

# RUSCH® FILIFORMS AND FOLLOWERS

## PRODUCT FEATURES

- ① Disposable
- ② Spiral or Straight Tip Filiforms
- ③ Bougie or Catheters Followers
- ④ Full Range of Sizes

### FILIFORMS

PRODUCT NUMBER	DESCRIPTION	SIZE
342103	Filiform Straight Tip	3 Fr.
342104	Filiform Straight Tip	4 Fr.
342105	Filiform Straight Tip	5 Fr.
342106	Filiform Straight Tip	6 Fr.
343103	Filiform Spiral Tip	3 Fr.
343104	Filiform Spiral Tip	4 Fr.
343105	Filiform Spiral Tip	5 Fr.
343106	Filiform Spiral Tip	6 Fr.

### BOUGIE FOLLOWERS

PRODUCT NUMBER	DESCRIPTION	SIZE
345108	Phillips Bougie Follower	8 Fr.
345110	Phillips Bougie Follower	10 Fr.
345112	Phillips Bougie Follower	12 Fr.
345114	Phillips Bougie Follower	14 Fr.
345116	Phillips Bougie Follower	16 Fr.
345118	Phillips Bougie Follower	18 Fr.
345120	Phillips Bougie Follower	20 Fr.
345122	Phillips Bougie Follower	22 Fr.
345128	Phillips Bougie Follower	24 Fr.

### CATHETER FOLLOWERS

PRODUCT NUMBER	DESCRIPTION	SIZE
346108	Phillips Catheter Follower	8 Fr.
346110	Phillips Catheter Follower	10 Fr.
346112	Phillips Catheter Follower	12 Fr.
346114	Phillips Catheter Follower	14 Fr.
346116	Phillips Catheter Follower	16 Fr.
346118	Phillips Catheter Follower	18 Fr.
346120	Phillips Catheter Follower	20 Fr.
346122	Phillips Catheter Follower	22 Fr.
346124	Phillips Catheter Follower	24 Fr.
346126	Phillips Catheter Follower	26 Fr.
346128	Phillips Catheter Follower	28 Fr.
346130	Phillips Catheter Follower	30 Fr.

TO PLACE AN ORDER CONTACT CUSTOMER SERVICE AT 1.866.246.6990

Teleflex and Rusch are trademarks or registered trademarks of Teleflex Incorporated or its affiliates.  
© 2013 Teleflex Incorporated. All rights reserved. 2012-1430