



# Vasc Band™ Hemostat


The Clear Difference in Hemostasis

# Inflatable Compression for Access Site Hemostasis

**Detachable strap**  
secures syringe during  
patient transport

**Extended hook &  
loop retention straps**  
provide quick application and  
enhance patient comfort

**Clear plastic band**  
provides access  
site visualization

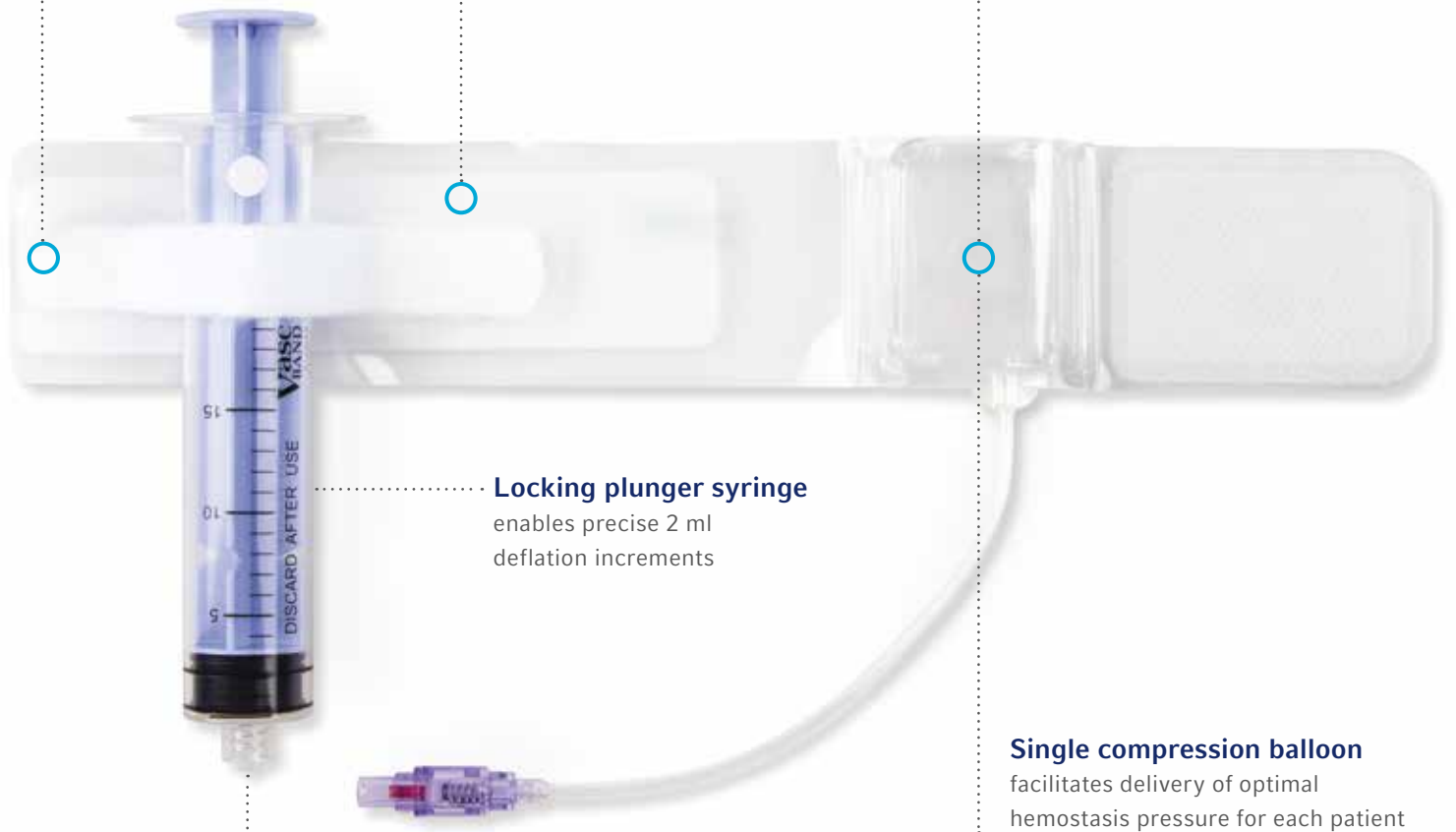

**Locking plunger syringe**  
enables precise 2 ml  
deflation increments


**Single compression balloon**  
facilitates delivery of optimal  
hemostasis pressure for each patient



**Secure twist connector**  
provides easy and reliable connection  
between syringe and inflation valve

**Five band sizes**  
Short - 21 cm  
Regular - 24 cm  
Long - 27 cm  
Extra Long - 29 cm  
XXL - 37 cm

The Vasc Band™ Hemostat delivers more direct radial compression than the standard inflatable radial compression device using recommended clinical protocols, with minimal compression of the ulnar artery.\*

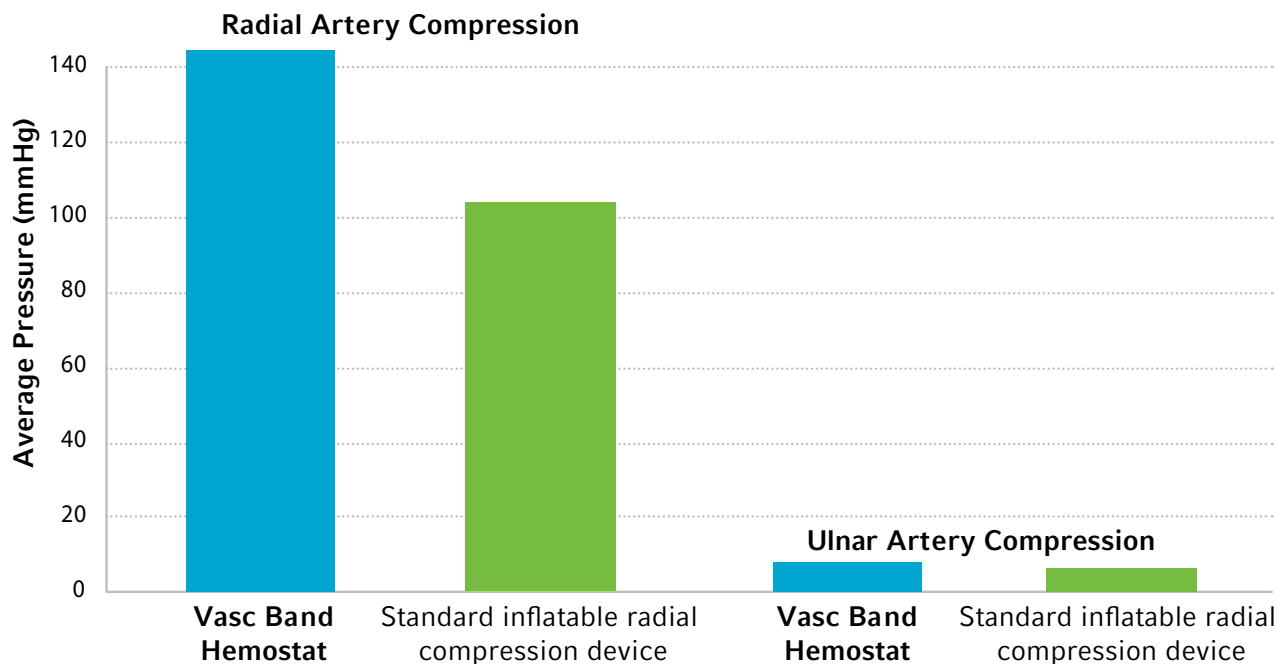

**Test protocol:** The Vasc Band Hemostat and the standard inflatable radial compression devices were tested on 10 human subjects. For each test, the compression band was secured to the wrist and inflated with recommended nominal and maximum inflation values for each device. Per the product IFUs, nominal air inflation is 15 ml for the Vasc Band Hemostat and 13 ml for the standard inflatable radial compression device, while maximum air inflation of 18 ml is the same for both devices. Pressure applied by each device to the radial and ulnar arteries was recorded using a Tekscan® ELF Force and Load Measurement System and FlexiForce® B201 flexible sensors. Data shown in the chart reflects maximum inflation value of 18 ml for both devices.

\*Data on file at Teleflex. Comparative data may not be indicative of clinical performance.

## Ordering Information

### Vasc Band™ Hemostat

The Vasc Band™ Hemostat is a compression device to assist hemostasis of arterial, venous and hemodialysis percutaneous access sites.

MODEL	BAND SIZE
3521	Short - 21 cm
3524	Regular - 24 cm
3527	Long - 27 cm
3529	Extra Long - 29 cm
3537	XXL - 37 cm

Packaged in quantities of 10 units per box. Individual units are not sold separately.

Teleflex is a global provider of medical technologies designed to improve the health and quality of people's lives. We apply purpose driven innovation – a relentless pursuit of identifying unmet clinical needs – to benefit patients and healthcare providers. Our portfolio is diverse, with solutions in the fields of vascular and interventional access, surgical, anesthesia, cardiac care, urology, emergency medicine and respiratory care. Teleflex employees worldwide are united in the understanding that what we do every day makes a difference. For more information, please visit [teleflex.com](http://teleflex.com).

Teleflex is the home of Arrow®, Deknatel®, Hudson RCI®, LMA®, Pilling®, Rüschi® and Weck® – trusted brands united by a common sense of purpose.

### Corporate Office

Phone +1 610 225 6800, 550 E. Swedesford Road, Suite 400, Wayne, PA 19087, USA

### Regional Offices

**United States:** Phone +1 919 544 8000, Toll Free 866 246 6990, [cs@teleflex.com](mailto:cs@teleflex.com), 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

**Latin America:** Phone +1 919 433 4999, [la.cs@teleflex.com](mailto:la.cs@teleflex.com), 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

**International:** Phone +353 (0)9 06 46 08 00, [orders.intl@teleflex.com](mailto:orders.intl@teleflex.com), Teleflex Medical Europe Ltd., IDA Business and Technology Park, Dublin Road, Athlone, Co Westmeath, Ireland

**Australia** 1300 360 226

**Austria** +43 (0)1 402 47 72

**Belgium** +32 (0)2 333 24 60

**Canada** +1 (0)800 387 9699

**China (Shanghai)** +86 (0)21 6163 0965

**China (Beijing)** +86 (0)10 6418 5699

**Czech Republic** +420 (0)495 759 111

**France** +33 (0)5 62 18 79 40

**Germany** +49 (0)7151 406 0

**Greece** +30 210 67 77 717

**India** +91 (0)44 2836 5040

**Italy** +39 0362 58911

**Japan** +81 (0)3 6632 3600

**Korea** +82 2 536 7550

**Mexico** +52 55 5002 3500

**Netherlands** +31 (0)88 00 215 00

**New Zealand** 0800 601 100

**Poland** +48 22 4624032

**Portugal** +351 22 541 90 85

**Singapore** (SEA non-direct sales countries) +65 6439 3000

**Slovak Republic** +421 (0)3377 254 28

**South Africa** +27 (0)11 807 4887

**Spain** +34 918 300 451

**Switzerland** +41 (0)31 818 40 90

**United Kingdom** +44 (0)1494 53 27 61

**For more information, please visit [teleflex.com](http://teleflex.com).**

Please see the Instructions for Use for a complete listing of the indications, contraindications, warnings and precautions. CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

Teleflex, the Teleflex logo, Arrow, Deknatel, Hudson RCI, LMA, Pilling, Rüschi, Vasc Band, and Weck are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. Other names may be trademarks of their respective owners.

Information in this material is not a substitute for the product Instructions for Use. Not all products may be available in all countries. Please contact your local representative. Revised: 04/2018.

© 2018 Teleflex Incorporated. All rights reserved. MC-004082 · Rev 0